



**THE HONG KONG STANDARDS AND TESTING CENTRE LTD.**  
**EC DECLARATION OF CONFORMITY**

**Reference Number:** EMC0704459DOC

Taising Hearing Amplifier Limited  
Flat C, 15/F., Phase 1, Golden Dragon Ind. Ctr.,  
152-160 Tai Lin Pai Road,  
Kwai Chung, N.T., Hong Kong

Description: Deluxe Amplifier II  
Brand Name: N/A  
Model: TS-2328

complies with the requirements of the  
**EC Council Directive on electromagnetic compatibility 2004/108/EC**

**Applicable Standard(s) with amendments:**

EN55013: 2001 +A1: 2003 +A2: 2006  
EN55020: 2007

**General Remarks:**

This declaration is only valid when used in conjunction with the technical file(s) refers to HM160006

This declaration applies specifically to the sample(s) investigated in the technical report mentioned above, and not to the bulk

The CE marking as shown below can be affixed on the product after preparation of necessary conformity documentation, as stipulated in Articles of the Council Directive 2004/108/EC

**Manufacturer's Representative**


**Signature**



**Test Laboratory**



ElectroMagnetic Compatibility Department  
For and on behalf of

The Hong Kong Standards and Testing Centre Ltd

www.hkstc.org


**Date of Issue:** 2007-09-13

**Deluxe** Amplifier II™  
PLUS TONE CONTROL ADJUSTABLE

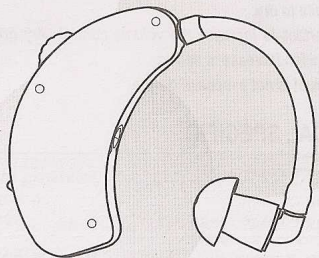

TS-2328

User Manual

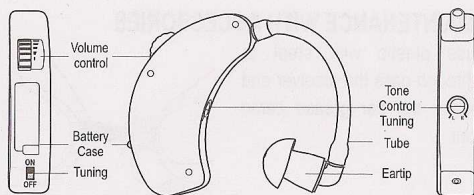


**INSTRUCTIONS FOR USE**

Turn the volume control to low level.  
Check the battery is in place.  
Place the hearing into your ear left or right.  
Turn the switch to ON. Increase volume control as your suitable level. (Don't use your hand cover the unit, it will be made the high noise pitch) Also can be use plastic stick to adjust the Tone Control Tuning.

**ADJUSTABLE TUBE LENGTH AND EARTIP DIRECTION**

Turn the eartip to the appropriate position for left or right ear and adjust the length. (If feel too long, can be take out eartip to cut the tube for your length. After cut the tube replace your eartip into tube)

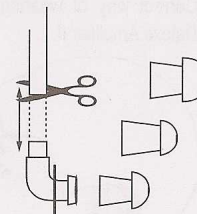

**ON / OFF SWITCH**


The correct positions of the switch are as shown below.

The switch should be turned to the "OFF" position when the apparatus is not in use. This will ensure longer battery life.



**VOLUME CONTROL**


There are volume control from LOW to HIGH.

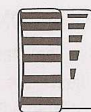

**BATTERY INSTALLATION**



Open the battery hatch with the finger tip and insert the battery as shown (Use 1.5V LR-44 battery).

Battery direction must be insert in right postion.

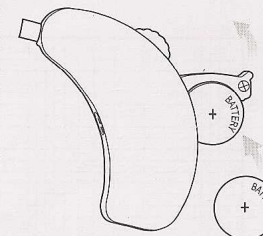


### TONE CONTROL TUNING

Use the plastic stick end to adjust the Tone Control Tuning to adjust as "L" Low & "H" High.

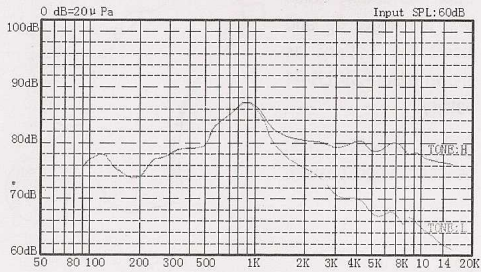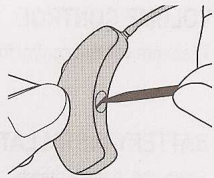

### CORRECT WAY TO WEARING AND USE


Correct way of wearing Deluxe Amplifier II.

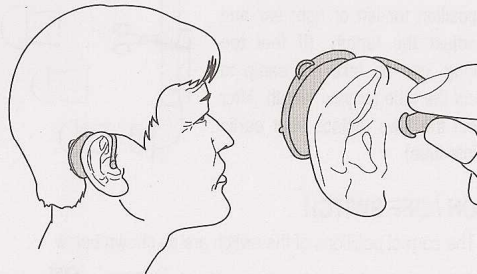


Eartip must be inserted deeply into the ear. Make sure ear tip is tight and the tips is pushed fully on the unit.

### MAINTENANCE WITH ACCESSORIES

use plastic with steel to through pass the receiver end make the ear grease came out.

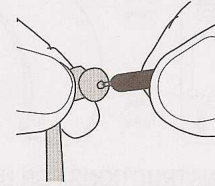

Use a soft brush to brush gentle back and across the receiver end remove the ear grease. This prevent ear grease buildup at receiver end.

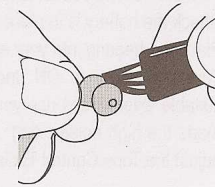

### CAUTION

1. The products is not water proof therefore the users must take care to prevent water form entering the unit. Should water enter the Deluxe Amplifier II remove the battery and place the unit in a warm place to dry.
2. It is important to increase the volume control very gradually to avoid sudden increase in sound.
3. This produst is not a medical device.

### TECHNICAL SPECIFIC

Battery Life time	120~180Working Hours(AG13 1PC)
Operating Current	0.8~3.0mA
Operating Voltage range	1.0~1.6V
Weight(include battery)	8.5g
Frequency Band Width	100~10K(Hz)+6dB
Full-on Gain dB	97dB
Harmonic distortion%	<2.6%
Equivalent input noise Level	<43dB