


# incredible

TS-2008

# ear amplifier

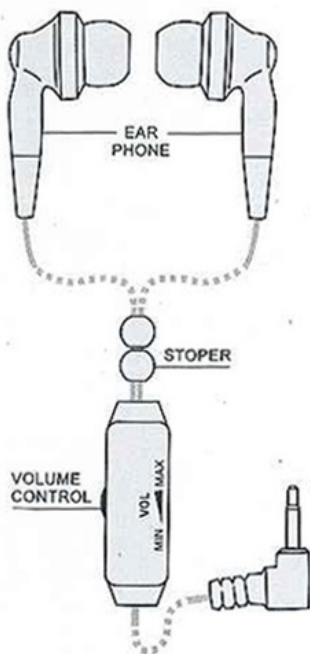


## Instructions For Use

1. CHECK THE BATTERY IS INPLACE (SEE BATTERY INSTALLION).
2. PLACE EAR PHONE INTO YOUR EARS.
3. TURN THE VOLUME CONTROL TO "MIN."
4. TURN "POWER SWITCH" TO "I" POSITION. WHEN THE APPARATUS IS NOT IN USE, TURN TO THE "O" POSITION, THIS WILL BE ENSURE LONGER BATTERY LIFE.
5. INCREASE VOLUME CONTROL TO YOUR SUITABLE LEVEL.

## TS-2008 Electrical Characteristic

Battery Life time	280 ~ 500 Working Hours(A675 1PC)
Operating Current	0.8 ~ 2.5 mA
Operating Voltage range	1.0 ~ 1.6 V
Frequency Band Width	100 ~ 10 K(Hz) $\pm$ 5 dB
SSPL90dB	105 dB
Full-on Gain dB	95 dB
Harmonic distortion%	$\leq$ 2.2%
Equivalent input noise Level	$\geq$ 50 dB

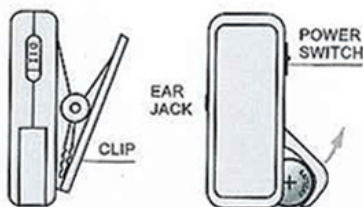

## Battery Installation

Open the battery hatch with the finger tip and insert the battery as shown (Use A675 zinc air battery). When begin place battery, tear out the paper. Zinc air battery is need to fill the air into battery before use. Battery direction must be insert in right postion.

## CAUTION

1. The products is not water proof therefore the users must take care to prevent water form entering the unit. Should water enter the Incredible Ear Amplifier remove the battery and place the unit in a warm place to dry.
2. It is important to increase the volume control very gradually to avoid sudden increase in sound.
3. This product is not a medical device.